

Additional information



Driver indicator lights:

On the top edge of the driver, colour indicator lights are displayed representing the various functions.

Green = Infusion

Yellow = Aspiration

Flashing Red = System over pressure warning

Steady Red = End of driver life

To remove the syringe from the driver:

Press the square coloured button on the side of the driver. The syringe will release from the driver.

End of driver life indicator:

Once the end of driver life indicator on the driver (steady red light) comes on, the component should be disposed of in accordance with hospital procedure. Do not start a procedure if the end of driver life indicator is already on. If the end of driver light indicator lights up during a procedure, immediately disconnect the syringe and either replace the driver or complete the procedure manually.

KEY POINTS TO REMEMBER

- Remove the black cap off the driver before use and dispose
- Never store the driver with a syringe attached

Comprehensive Instructions for Use are available via the Medovate website as well as being supplied with the product.

Please read the Instructions for Use before using SAFIRA® for the first time.

MEDOVATE
Developing Innovation

medovate.co.uk

Email: enquiries@medovate.co.uk

X :@Medovate

LinkedIn: Medovate Limited

SAFIRA®
Giving Anaesthetists Control

SAFIRA® QUICK START GUIDE



Quick start: the SAFIRA® system

IMPORTANT: Please ensure the black protective foam cap is removed from the syringe driver and disposed.



Please ensure all equipment is clean (if not brand new) and all relevant expiration checks are carried out in line with local procedures.

